Hydrogel Pads

Instructions for use Gebruiksaanweisung

Mode d'emploi

Istruzioni d'uso

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EN

Instructions for use

The Hydrogel Pads are intended for single use as soothing breast pads for nursing mothers.

- Wash hands before using the product.
- Open pouch.
- Remove gel pad from the plastic liner.
- Gently place the gel side of the pad directly over your nipple.
- Do not use if the pouch is open or damaged.
- Prior to breastfeeding, remove the gel pad.
- Place on a clean surface.
- Gently wash the breast with warm water and pat dry.
- Then start breastfeeding.
- After breastfeeding, utilize the gel pad.

Do not use soaps or detergents to clean pad • Store gel pads in a cool, dry place • For more cooling sensation, gel pads can be stored in the fridge • Reusable for 24 hours • If gel pad becomes cloudy or milky, please discard and use a new one.
Optional: To improve soothing, briefly pass the gel side of the pad under a cool stream of water (1–2 seconds), shake off excess water and set aside for 2 minutes. Do not rinse more than twice a day.

Important Safeguards

Use only as indicated • Keep out of reach of children, especially the plastic liner • Extra care should be taken if using with cracked nipples or fungal infection • If nipples are cracked, bleeding or have a burning sensation, contact your lactation consultant or physician • To prevent nipple pain it is important that baby feeds at mother's breast in a correct position.
Ingredients: Water, glycerol, polymer

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© Medela AG 200.1726/2018-04/K