Spare Teats

Ideal for feeding breast milk

Note: Only use Medela original accessories.

Make sure breastfeeding is well established before introducing a teat.

Cleaning and Hygiene

- I Drinking water quality for any of the following described cleaning procedures.
- I Clean parts immediately after use to avoid dry drop of milk residues and to prevent from growth of bacteria.
- I Take care not to damage parts when using a brush.
- I Inspect all items before each use.
- I Replace parts if damaged or worn out.
- I For hygiene reasons, we recommend replacing teats after 3 months.
- I Note: Food colorings may discolor components which will not affect performance.
- I Follow these cleaning instructions unless you are told otherwise by your healthcare professional.
- I Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant („sterilizing solution“) for longer than recommended, as this may weaken the teat.

Prior to first use and once daily

- I Disassemble, wash and rinse all parts.
- I Place parts in a saucupan of sufficient capacity.
- I Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- I Heat it up and allow to boil for 5 minutes.
- I Remove from stove and drain water off.
- I Dry parts with a clean towel.

As an alternative to boiling: Dishwasher: Clean parts in top rack.

Microwave: Use Medela Quick Clean™ Microwave Bags or other microwave-safe device.

Steam sterilizer: Follow the manufacturer's instructions.

After each use

- I Disassemble and wash all parts in warm, soapy water.
- I Rinse in clear water. Dry parts with a clean towel.
- I After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- I Alternatively, the parts can be wrapped in a clean paper or cloth towel.
- I Do not paper or clothe in direct sunlight.
- I Avoid touching the inside of bottles and lids.

For your child's safety and health

WARNING!

- I Always use this product with adult supervision.
- I Never use feeding teats as a sootheer.
- I Continuous and prolonged sucking of fluids will cause tooth decay.
- I Always check food temperature before feeding
- I Keep all components not in use out of the reach of children.
- I Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Material Silicone
Disposal According to local regulations
Conforms to EN 14350.

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